

Mediana A16 AED

Function Overview Guide

The A16 is designed with clear controls, visual indicators, and guided operation. This overview helps buyers and resellers quickly understand the device layout, key functions, and readiness features that support AED placement and use.

MicroSD Card Port

Battery
Non-Rechargeable
LiMnO2

IR Port

Adult/ Pediatric Pads

Can be used
as 2-in-1 Pads

CPR
Feedback



Device Configuration

1 Connecting Socket
Defibrillator pads connector socket.
The pads are always ready for use.

2 Language Button
User can select the desired language
among 3 pre-installed languages.

3 Speaker
Acoustic indicator for real
time guidance.

**4 Adult/Pediatric Mode
Switch Button**
Easily switch from adult to pediatric
mode without changing pads.

5 Power Button
The device will be activated.

6 Voice Recording
On-site recording function.

7 Pad Connect Icon
Guidance on connecting pads.

**8 Pads Attachment
Icon**
Guidance on where to attach the pads.

9 Analysing ECG Icon
Guidance not to touch patient
during ECG analysis.

10 Shock Button
Push the flashing "Ready for shock
delivery" button to deliver shock.

11 CPR Icon
Guidance of the CPR administration
with metronome after electric shock.

12 Status Indicator
AED status, Battery level status,
Temperature status, Pads status.

