

Mediana A16 AED

Quick Use Instruction

The Mediana HeartOn A16 is designed for simple, step-by-step operation. Follow the voice prompts and on-screen illustrations.

Basic 4-step Operation



1 Open and Power On

Open the cover and press the "Power On" button.



2 Attach the Pads

Apply the pads as illustrated.



3 Analyze and Charge

The device analyzes heart rhythm and charges simultaneously to reduce time before shock delivery.



4 Press the Shock Button

For semi-automatic operation, press the shock button only when directed by the AED.



Always follow device prompts, user's manual and applicable emergency procedures.

Voice prompts and metronome, help guide responders through CPR and AED operation.

